


3M Curos^m

Disinfecting Cap for Needleless Connectors

Disinfects in one minute – protects needleless connectors for up to seven days.

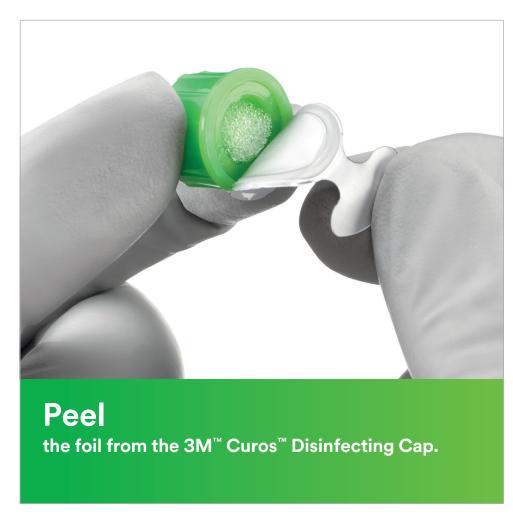



Push and twist

the Curos disinfecting cap clockwise onto the end of the needleless connector.

- ▶ If needleless connector is heavily soiled, clean with 70% isopropyl alcohol swab before using cap
- ▶ Do not use cap if seal is broken, torn, punctured, or sponge appears dry
- Cap must remain on needleless connector for a minimum of one minute and may remain on for seven days if not removed
- ► The needleless connector can be accessed immediately after removing the Curos disinfecting cap that has been in place for a minimum of one minute
- > Always place a new Curos disinfecting cap on needleless connector after each use
- Dispose of the Curos disinfecting cap after every use
- Follow hospital protocol for disinfecting needleless connectors, if performing sequential accesses

<u>Warning:</u> to avoid potential for injury, use only on needleless connectors. <u>Caution:</u> potential choking hazard.

Active ingredient: 70% isopropyl alcohol (IPA). Single use only. This device is not made with natural rubber latex or plasticiser Diethylhexylphthalate (DEHP). Refer to current device labeling for indications for use, warnings and product details.

