

**AQUAFIBER EXTRA**

**WOUND CARE  
BECAUSE YOU CARE**

## FEATURES

Highly absorbent // High gelling // The dressing can remain *in situ* for up to 7 days // Reduces risk of maceration // Maintains a moist wound healing environment // Aids autolytic debridement // High wet tensile strength //

# ACTIVHEAL® AQUAFIBER EXTRA CREATES A MOIST WOUND HEALING ENVIRONMENT FOR MODERATE TO HEAVILY EXUDING WOUNDS.

ActivHeal® Aquafiber Extra is an innovative combination of two proven wound care technologies based on CMC & alginate fibres. A hidden integrity layer in the dressing supports the tensile performance.


ActivHeal® Aquafiber Extra is a soft, conformable, highly absorbent and non-woven dressing which turns into a soft, cohesive gel when in contact with wound exudate. Thus it helps with autolytic debridement and maintains a moist wound healing environment following the principles of advanced wound care. ActivHeal® Aquafiber Extra effectively addresses the key challenges of improved exudate management, gelling & tensile performance.

- ✓ HIGH ABSORBENCY<sup>1</sup>
- ✓ HIGH WET TENSILE STRENGTH<sup>1</sup>
- ✓ INCREASED VERTICAL WICKING<sup>1</sup>

## PERFORMANCE



### High absorbency

The unique design & fibre engineering of ActivHeal® Aquafiber Extra allows the dressing to absorb exudate & transform into a cohesive gel.

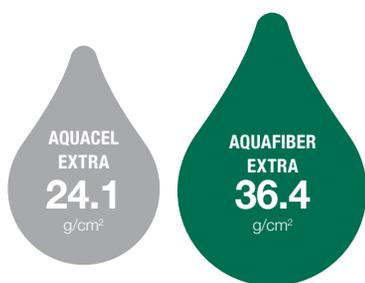

### High wet tensile strength

The quilting pattern of ActivHeal® Aquafiber Extra is supported by a hidden integrity layer allows for intact removal of the dressing.

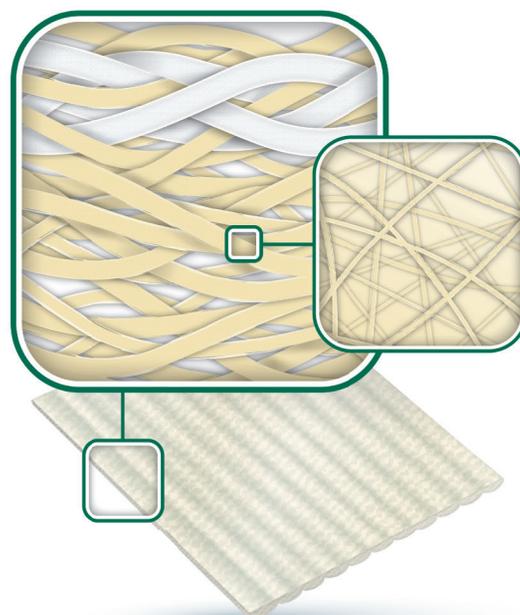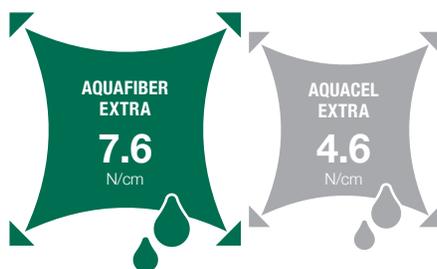

### Low lateral wicking

In-house analysis<sup>1</sup> has confirmed that the vertical wicking of the ActivHeal® Aquafiber Extra can be potentially helpful in the prevention of maceration within the peri-wound skin.

## INDICATIONS


ActivHeal® Aquafiber Extra is indicated for the management of the following moderately to heavily exuding acute and chronic wounds:

- Pressure ulcers
- Venous leg ulcers
- Arterial leg ulcers
- Diabetic ulcers
- Graft and donor sites
- Post-operative surgical wounds
- Cavity wounds
- Superficial and partial thickness burns

### Moderate > Heavy Exudate



### Wound Types

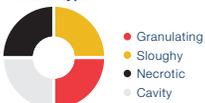

## SIZES AND CODES

### ACTIVHEAL® AQUAFIBER EXTRA

SIZE (CM)	QTY PER CARTON	QTY PER CASE	PRODUCT CODE	NHS SUPPLY CHAIN CODE	DT CODE
5x5	10	10	9023625	ELY795	407-5339
10x10	10	10	9023632	ELY796	407-5347
15x15	5	10	9023649	ELY797	407-5628
20x30	5	10	9023656	ELY798	407-5321
2.5x30.5 Ribbon	5	10	9023663	ELY799	407-5602
2x46 Ribbon	5	10	9023670	ELY800	407-5610

ActivHeal® Aquafiber Extra is available through the NHS Supply Chain and Drug Tariff

## Advanced Medical Solutions Ltd

Premier Park, 33 Road One, Winsford Industrial Estate, Winsford, Cheshire CW7 3RT  
 T 08444 125 755 E [activheal@admedsol.com](mailto:activheal@admedsol.com) W [www.activheal.com](http://www.activheal.com)



References 1. AMS Data on file LD024-16, LD037-16.

ActivHeal is a Registered Trademark of Advanced Medical Solutions. Aquacel Extra is a registered trademark of ConvaTec.

© Advanced Medical Solutions 2018 // Review date May 2020 // Document code AMS/AH-012-01