

**ACTIVHEAL® SILICONE ADHESIVE FOAM LITE IS A STERILE WOUND DRESSING THAT IS SOFT, THIN, DISCRETE AND CONFORMABLE TO ANATOMICAL CONTOURS.**


- ✓ Breathable
- ✓ Gentle secure adhesion to the skin
- ✓ Can remain in situ for 7 days
- ✓ Prevents pain at dressing changes
- ✓ Perforated wound contact layer
- ✓ Protects the periwound
- ✓ Water resistant and bacterial barrier
- ✓ Low friction
- ✓ Easy to remove
- ✓ Provides a moist wound environment



## SILICONE FOAM LITE DRESSING RANGE

ActivHeal® Silicone Foam Lite dressings are breathable and designed to manage non to low exuding wounds. The dressing is easy to remove and provides a gentle but secure adhesion to skin with minimal epidermal stripping or pain during removal.

ActivHeal® Silicone Foam Lite dressing has a three layer structure consisting of a hydrophilic, absorbent, polyurethane central foam layer, and a breathable and low friction, polyurethane film outer layer, which provides a waterproof and bacterial barrier. The wound contact layer is a silicone adhesive coated perforated film.

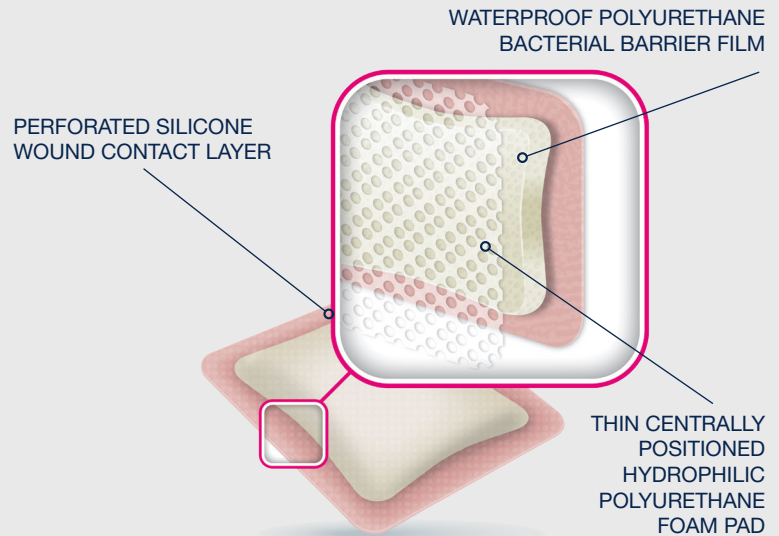

### Indications for Use

ActivHeal® Silicone Foam Lite dressing is indicated for use in the management of non to low exuding, chronic and acute wounds throughout the healing process. The dressing is indicated for use on the following wounds:

- ✓ Skin Tears
- ✓ Post Operative Surgical Wounds
- ✓ Superficial and Partial Thickness Burns
- ✓ Abrasions and Lacerations
- ✓ Pressure Injuries
- ✓ Venous Leg Ulcers
- ✓ Arterial Leg Ulcers
- ✓ Diabetic Ulcers
- ✓ Radiation Damaged Skin

### Exudate Level

Nil > Low  
Exudate



### Wound Types

- Fungating
- Cavity
- Granulating
- Sloughy
- Necrotic



ActivHeal® Silicone Foam Lite dressings are suitable for use under compression bandaging.

### Ordering Information

#### ActivHeal® Silicone Foam Lite Border

5 x 5cm	ADM9023687	Box/10
5.5 x 12cm	ADM9023694	Box/10
7.5 x 7.5cm	ADM9023700	Box/10
10 x 10cm	ADM9023717	Box/10
15 x 15cm	ADM9023724	Box/10
10 x 20cm	ADM9023731	Box/10
10 x 30cm	ADM9023748	Box/10

#### ActivHeal® Silicone Foam Lite Non-Border

7.5 x 7.5cm	ADM9023755	Box/10
10 x 10cm	ADM9023762	Box/10
15 x 15cm	ADM9023779	Box/10
10 x 20cm	ADM9023786	Box/10

Discover how ActivHeal® can offer your practice a simple, clear and cost effective wound care range, ensuring the best clinical outcome for the patient.



Contact your Team Medical Representative for product information and pricing or to arrange a product evaluation.

### Team Medical Supplies

Order online at: [www.teammed.com.au](http://www.teammed.com.au) | Order via email: [orders@teammed.com.au](mailto:orders@teammed.com.au)

Order via phone: 1300 22 44 50 | Order via fax: 1300 22 44 60