



TESTPACK™ PLUS HCG URINE WITH OBC
TESTPACK™ PLUS HCG COMBO WITH OBC

RESULTS YOU CAN COUNT ON

For the Rapid Detection of Early Pregnancy



TESTING WITH CONFIDENCE

TestPack™ Plus hCG offers highly sensitive test results in an easy-to-read “+” or “-” format. With the added benefit of unique on-board quality controls and an indicator telling you when it is time to read test results, you can have confidence in every test performed.

Unlike other rapid tests that only incorporate a procedural control, TestPack™ Plus hCG tests have built-in positive and negative controls, which ensure that the test is operating properly:

- Assures validity of results
- Monitors sample flow
- Negative control checks for interfering substances

TestPack™ Plus hCG is an ideal choice for any healthcare setting — hospital, laboratory or point of care:

- The only test with on-board positive and negative controls for greater confidence in test results
- Results in “+” or “-” for simple result interpretation
- Over 99% sensitivity and specificity for trusted clinical performance
- Available in urine or serum* test formats for flexible sampling

ACCURATE RESULTS YOU CAN TRUST

Both TestPack™ Plus hCG Urine and TestPack™ Plus hCG Combo have shown in studies to have a relative sensitivity and relative specificity of over 99%.

		TESTPACK™ PLUS HCG URINE WITH OBC	
		+	-
ANOTHER COMMERCIAL HCG COMBO TEST	+	131	0
	-	0	169

		TESTPACK™ PLUS HCG COMBO WITH OBC		TESTPACK™ PLUS HCG COMBO WITH OBC (SERUM)	
		+	-	+	-
ANOTHER COMMERCIAL HCG COMBO TEST	+	126	1	110	0
	-	0	168	0	76



SENSITIVITY

TestPack™ Plus hCG can detect human chorionic gonadotropin (hCG) in samples at concentrations of 25 mIU/mL or greater.



SPECIFICITY




TestPack™ Plus hCG tests have been successfully evaluated to show no cross-reactivity with a variety of substances, including other hormones that can be found in urine and serum.*

EASY TESTING PROCESS

1 Collect the sample (urine or serum*).




2 Dispense into the sample well.

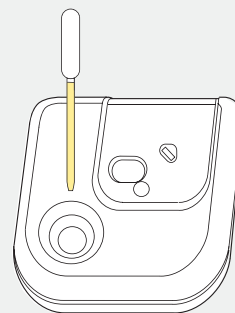


3 Read the test results.

POSITIVE TEST RESULTS

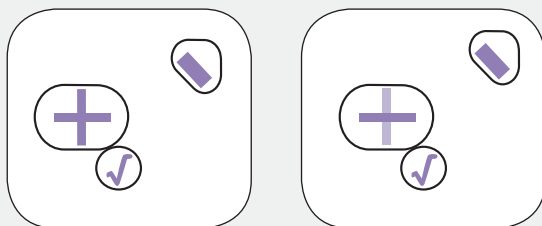

The appearance of a "+" symbol indicates a positive test result.

NEGATIVE TEST RESULTS

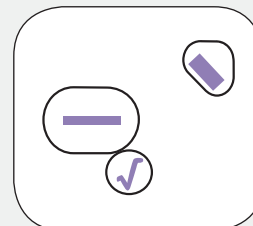

The appearance of a "-" symbol indicates a negative test result.


PRECISION YOU CAN SEE



PLUS/MINUS RESULTS
Easy-to-read test results

END-OF ASSAY INDICATOR
Tells you when the test is ready to read
(TestPack Plus hCG Urine only)

NEGATIVE CONTROL
Indicates the possibility of an interfering substance, causing an invalid result

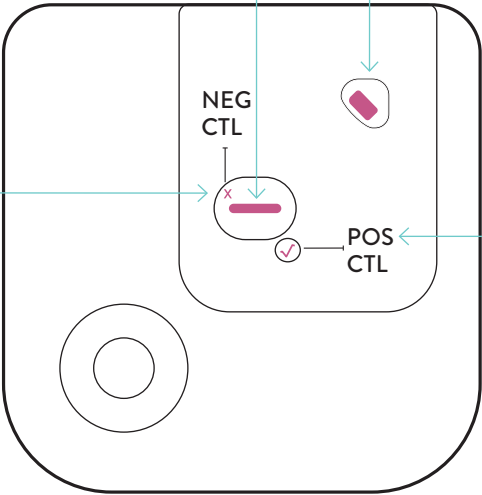

POSITIVE CONTROL
Present in all valid tests, whether hCG is present or not

PRODUCT INFORMATION

	TESTPACK™ PLUS HCG URINE WITH OBC	TESTPACK™ PLUS HCG COMBO WITH OBC
Catalogue Number	505798J	505805J
Kit Size	20 Tests	20 Tests
Storage	2°C–30°C	2°C–30°C
Sample Type	Urine	Urine or Serum
Time to Result	At End of Assay	5 Minutes
Sensitivity	25 mIU/mL hCG	25 mIU/mL hCG
Specificity	100%**	> 99%**

*Serum sampling only available for TestPack™ Plus hCG Combo with OBC.
**Relative sensitivity and specificity compared to another professional pregnancy test.

TO LEARN MORE, CONTACT YOUR LOCAL ABBOTT REPRESENTATIVE OR VISIT [GLOBALPOINTOFCARE.ABBOTT](https://globalpointofcare.abbott)

T: 1800 622 642 (free call AU) | T: 0800 466 951 (free call NZ) | T: +61 7 3363 7100 | rapiddx.ANZ.enquiries@abbott.com

