



QUICK, ACCURATE, EASY

# BIOLINE™ INFLUENZA ANTIGEN TEST

Detect all influenza subtypes  
in less than 15 minutes at the  
point-of-care.

The Bioline™ Influenza Antigen test is a rapid chromatographic immunoassay for the differential and qualitative detection of influenza virus type A and type B antigens directly from nasal, throat and nasopharyngeal swab or nasal/nasopharyngeal aspirate specimens.



# Bioline™ Influenza Antigen Test

## OVERVIEW

- Differential detection of influenza virus type A & B
- Detects all influenza A subtypes including H1N1
- Flexible sample type - nasal, throat and nasopharyngeal swab or nasal/nasopharyngeal aspirate specimens
- Simple three step procedure
- Results in 15 minutes or less
- Highly sensitive and specific - 91.8% sensitivity and 99.0% specificity vs. viral cell culture and RT-PCR as gold standard
- Kit contains all key materials required to perform a test

## CLINICAL PERFORMANCE


VIRAL CULTURE AND/OR RT-PCR	BIOLINE INFLUENZA AG		COMMERCIAL INFLUENZA TEST	
	Positive	Negative	Positive	Negative
Type A H3	81	4	73	8
Type A H5	12	2	9	3
Type B	41	5	38	3
Sub total	134	11	120	14
Negative	287	284	5	282
<b>TOTAL</b>	<b>421</b>	<b>295</b>	<b>125</b>	<b>296</b>
Sensitivity (95% CI)	91.8% (85.9 - 95.4%)		89.6% (83.2 - 93.7%)	
Specificity (95% CI)	99.0% (97.0 - 99.6%)		98.3% (96.0 - 99.3%)	

## TEST PROCEDURE (SWAB SPECIMENS)

- 1 Add Assay diluent to test tube.
- 2 Insert the patient swab specimen into the test tube. Swirl the swab at least five times. Squeeze swab upon removal.
- 3 Place the test strip into the test tube.

Consult the Instructions for use for complete information including the procedure for nasal/nasopharyngeal aspirate specimens and swab specimens extracted into transport media.

## INTERPRETATION

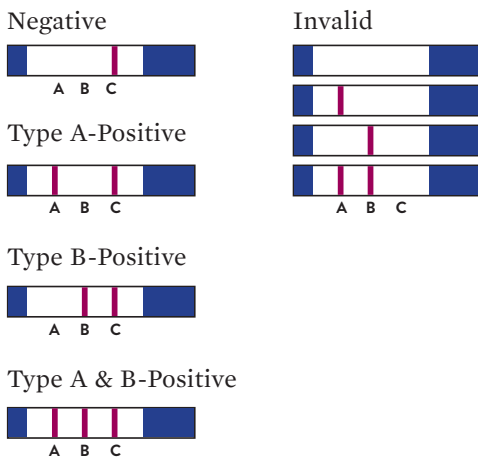

## ORDERING INFORMATION

PRODUCT NAME	ORDERING NO.
Bioline™ Influenza Antigen Test Kit (10 Tests)	19FK11
Bioline™ Influenza Antigen Test Kit (25 Tests)	19FK12

Kit contains: Test strips, Assay diluent, Disposable droppers, Sterilized swabs for specimen collection, Disposable test tubes, Control swabs, Instructions for use.

The Bioline™ Influenza Antigen Test Kit can only be purchased by healthcare professionals.

**TO LEARN MORE, CONTACT YOUR ABBOTT REPRESENTATIVE OR VISIT [ABBOTT.COM/POCT](https://www.abbott.com/poct).**

T: 1800 622 642 (free call AU) | T: 0800 466 951 (free call NZ) | T: +61 7 3363 7100 | [rapiddx.ANZ.enquiries@abbott.com](mailto:rapiddx.ANZ.enquiries@abbott.com)

© 2022 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners. Any photos displayed are for illustrative purposes only. Any person depicted in such photos is a model. COL-00828-02 03/22

