

# Getinge Assured

## Bowie-Dick Test Card TT Interpretation Guide


Sterilizer that is functioning normally should have an indicator with a uniform green colour change throughout the entire pattern.

### Pass

**Interpretation:** Complete colour change from purple to green, indicates that an effective steam sterilization cycle has been achieved.

### Pass

The chemical indicator card turns even brown in colour. The indicator has been subjected to a long drying phase.

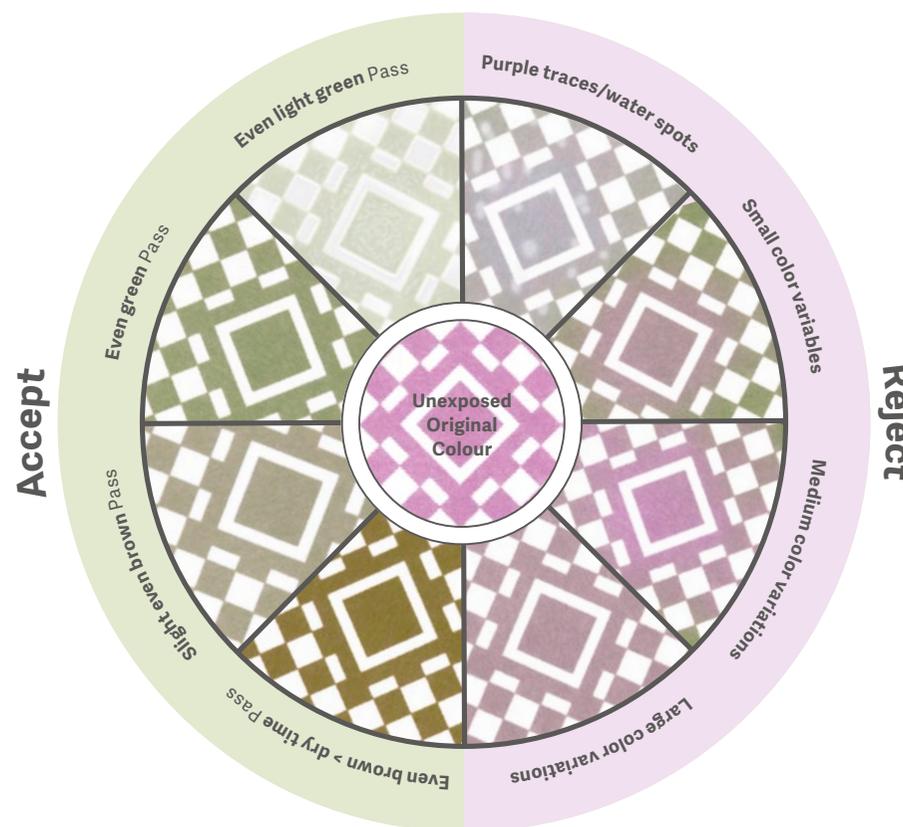

### Fail

Any residual purple colour indicates a fail.

Residual air is present in the sterilizer. It is recommended that the test be repeated to assure that the sterilizer is warmed-up and no anomalies are present in the single test cycle. If the same result is acquired the second time the sterilizer has failed the test.