

TITEMS Diagnostic Emulator

Type 6 Emulator Interpretation Guide

Pass



Pass: GREEN Different sterilizers produce varied results depending on quality of steam, as long as the emulator has turned **GREEN** the parameters have been met.

Unexposed



Fail



Fail: Any **PURPLE** or **GREY** on the indicator dot is a Fail. If in doubt do not use.

Wet Steam



Wet: Emulator laminate will “bubble” when excessive moisture has been present in the sterilizing process.

Superheated Steam



This is a **Fail**. Emulator will “burn” due to lack of moisture in the steam during sterilization.