Getinge Assured

Bowie-Dick Test Card TT

INTERPRETATION IMAGES

1

2

3

This is one example of a fail

DEUTSCH

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ENGLISH

Indications for Use

The Getinge Assured Bowie-Dick Test Card TT is for daily monitoring of small pre-vacuum type B steam sterilizers operating at 134°C. The card is specially packaged and designed to meet EN 867-5 requirements. This Getinge Assured product is tested and verified to have equivalent and approved performance at both EN 867-5 test cycles and Getinge Bowie-Dick cycles for type B pre-vacuum table top sterilizers.

Instructions for use

The test is conducted each day in an empty sterilizer chamber prior to processing goods.

1. Run a warm up cycle to properly heat the sterilizer.
2. Assemble the test by attaching the metal ring, provided with the indicator cards, through the hole on the top of the Bowie-Dick Test Card TT pouch. The ring provides stabilization to the pouch and indicator, which prevents it from being moved around inside the sterilizer chamber.
3. Hang the Bowie-Dick Test Card TT in an empty sterilizer.

4. Run a pre-vacuum steam sterilization cycle as specified by the sterilizer manufacturer. Note: a short dry time of 2 to 5 minutes will not affect the test.

Color change

1 Unprocessed 2 pass 3 Fail (any presence of purple indicates a fail)

Interpretation

1. Remove the test from the sterilizer as soon as the cycle has finished. Caution: The test pouch and ring may still be hot and should be handled with care.
2. Open the peel pouch and remove the Bowie-Dick Test Card TT. Retain the metal ring for later use and dispose the peel pouch.
3. If the sterilizer is functioning properly, the indicator graphic will show a uniform color change from purple to green throughout the entire indicator image.
4. A brown or non-uniform green color may indicate problems with steam conditions in the sterilizer chamber other than non-condensable gases or residual air. This may require a sterilizer service call.
5. Turn the Bowie-Dick Test Card TT over and fill in the appropriate information on the back. Save the card for file records.

Storage

Store at normal room temperature 10–38°C and 10–60% Relative Humidity. Protect from moisture and excess humidity.

Expiry Date

The expiry date is printed on the product packaging. Do not use after the expiration date on the product packaging.

Interfering Substances or Conditions

There are no known interfering substances or conditions that could affect the intended use of the indicator or adversely affect the indicator performance.

Release of Toxic Substances

The indicator releases no known toxic substances in sufficient quantities to cause either a health hazard, or a hazard to the intended properties of the product being sterilized before, during or after the sterilization process.

Declaration of Conformity

The Getinge Assured Bowie-Dick Test Card TT conforms to EN 867-5 performance requirements for Alternative Indicators Equivalent to the Porous Load Process Challenge Device Test.

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