



# Chamber Protect

## The MELAG chamber cleaning set

**Reliable cleaning and long-term protection that leaves your autoclave shining:**

The new MELAG chamber cleaning set removes even soiling that is invisible to the eye and thus saves the chamber from discolouration and deposits. Benefit from the Chamber Protect cleaning set when cleaning the chamber and trays once a week:


- ✓ Powerful cleaner with 50 lint-free cloths and 1 microfibre cloth
- ✓ Particularly gentle on the material of the chamber and door seal
- ✓ Approved through extensive material compatibility tests
- ✓ Ideally matched to all MELAG steam sterilizers



Art.Nr.: ME01081



## You have the choice:

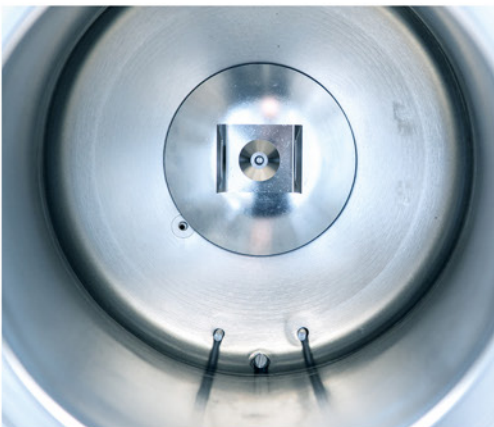

With regular use of  
Chamber Protect



Without regular  
chamber cleaning

# Step by step to the perfect result:

For a long-term protection and lasting shine, please apply the chamber cleaning set once a week as follows:




## 1. Apply Chamber Protect

Open the door of your autoclave, turn the device off and let it cool down. Next, apply approximately 2 sprays of Chamber Protect to the inside of the door and approximately 15 sprays into the chamber.



## 2. Spread Chamber Protect

Spread the cleaning spray over the entire stainless steel surfaces with one of the lint-free cloths. Allow the Chamber Protect spray to soak in for about 3 minutes.

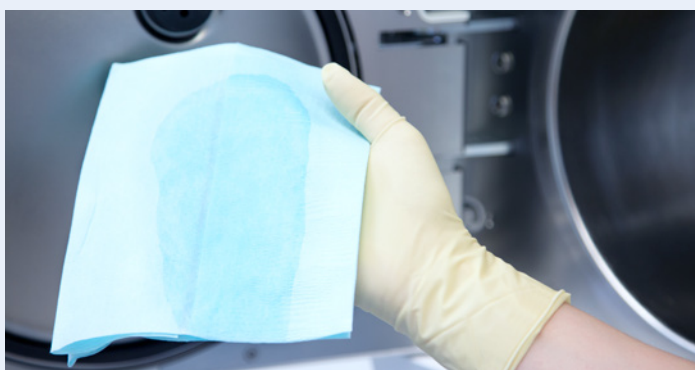

## 3. Clean the stainless steel surfaces

Take another lint-free cloth and moisten the cloth as well as the chamber with sufficient demineralised water (about 50 ml each). Wipe the surfaces sprayed with Chamber Protect with the cloth and allow it to soak in for about 5 minutes.



## 4. Dry the stainless steel surfaces

Finally, dry the inside of the door and the entire chamber thoroughly with the blue microfibre cloth. The autoclave is now available again for sterilizing the instruments.



For further information:  
[www.melag.com](http://www.melag.com)

**MELAG**  
competence in hygiene