





THE NEW GENERATION OF POST-OP WOUND DRESSING

The innovative level of **Leukomed® Control** reflects the philosophy and working practices of the **Leukoplast®** product range. Products should not just do their job, they should create optimal conditions for users and patients. **Leukomed® Control** scores highly on handling, efficiency and comfort. **Leukomed® Control** represents the latest generation of post-operative dressings.


How to apply **Leukomed® Control**

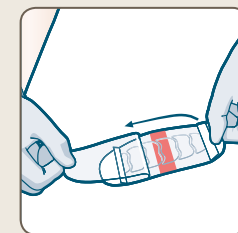


1. Remove the sterile film dressing from the packaging and tear off the transparent supportive film underneath.

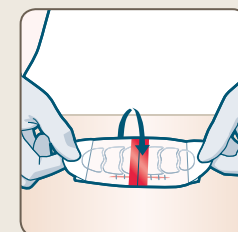


2. Hold the film dressing on both sides, turn the gel pads towards the wound and apply.

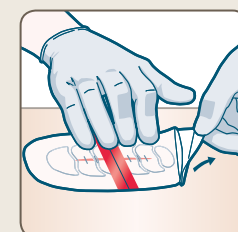

3. Tear off the white application support. Gently smooth the dressing down, so that the edges are wrinkle-free and stick securely.

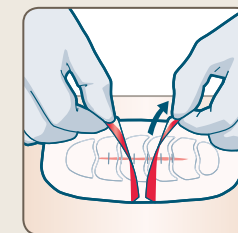


4. Carefully remove the two upper supports at the red indicators by pulling gently to the left and right.

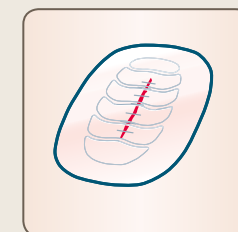

5. Check the wound regularly, without lifting the dressing.

BSN-VIDEO CHANNEL

You can find further details on the BSN medical video channel. Just follow the QR code below:

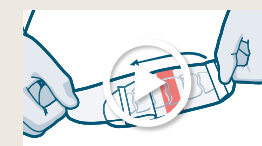

Leukoplast®
Like Skin

LEUKOMED® CONTROL

CLEARLY CONTROLLING THE RISK



04/2016
70651-00017-00

BSN medical GmbH

Quickbornstraße 24
20253 Hamburg
Deutschland
www.bsnmedical.com
Tel. +49 40 / 49 09-45 45
Fax +49 40 / 49 09-66 66

BSNmedical

THERAPIES. HAND IN HAND.

BSNmedical

LEUKOMED® CONTROL – MAXIMUM CONTROL

After surgery, the wound is observed especially closely. Infections must be quickly recognised and therapeutic measures rapidly taken. Thanks to the high transparency of the film and hydropolymer gel pad, with **Leukomed® Control**, wound inspection is possible at any time, without changing the dressing. This reduces the time expenditure of treatment and promotes healing by letting the wound rest.

PRODUCT DESIGN: THOUGHT OUT TO THE LAST DETAIL

The various components of **Leukomed® Control** make it a genuine specialist for post-operative wound care. Apart from the transparency of all the components, the hydropolymer gel pads and the multifunctional film offer particularly versatile benefits.

HYDROPOLYMER GEL PADS: THE BEST THING THAT CAN HAPPEN TO A WOUND

Owing to their reliable absorption capacity, the **Leukomed® Control** hydropolymer gel pads take up exudate from the wound and seal it in securely. Once applied, **Leukomed® Control** feels like a second skin, offering protection and an optimal moisture environment.

- Highly flexible, both lengthwise and widthwise
 - ▶ Optimally fits to body contours
 - ▶ Extremely comfortable, even in movement
- Provides an ideal wound environment
 - ▶ Supports the healing process

Quick mobilisation of patients

A declared aim of many caregivers is the rapid discharge of patients home or to rehab after surgery. **Leukomed® Control** supports these efforts:

- Transparency offers patients the possibility to observe their wound on their own
- Flexibility and elasticity promotes everyday coping
- High quality film allows showering and washing



A CRYSTAL-CLEAR PERSPECTIVE: EVERYTHING UNDER CONTROL



Leukomed® Control convinces by its high transparency, because only a well-controlled wound is a safe one. A change of dressing is no longer a matter of routine, but only necessary when there are signs of infection. Thus the advantages of **Leukomed® Control** are equally apparent to both caregivers and patients:

- Excellent wound observation
 - ▶ Rapid identification of surgery site infections (SSI)
- The patient can contribute to wound observation
 - ▶ High degree of compliance
- Less dressing changes
 - ▶ Less time-consuming
 - ▶ Low risk of contamination from the outside



MULTIFUNCTIONAL FILM – ALL INCLUSIVE



The multifunctional **Leukomed® Control** film is much more than just a dressing. Its high quality offers reliable protection from viruses, bacteria and water, at the same time as high water vapour permeability.

- Bacteria-proof
 - ▶ Protection from outside contamination
- Waterproof
 - ▶ Permits daily hygiene
- Allows maximum water vapour permeability
 - ▶ Reduces the risk of skin maceration
- Extra-thin and elastic
 - ▶ Very smooth
 - ▶ Comfortable to wear

SECURE ADHESIVE EDGE

- Extra-wide adhesive edge
 - ▶ Assures high durability
 - ▶ Permits daily hygiene



Indications



Leukomed® Control is indicated for dry to low exuding wounds, such as:

- Surgical wounds
- Traumatic wounds

Recommendation: It is especially suited for the control of wounds on predisposed patients with an increased risk of infection. Length of use up to seven days.

Ordering information Leukomed® Control

Ref-No.	Size	Pad Size	Items per Unit	Shipping Unit
73230-00	5 x 7 cm	2 x 4 cm	10	1/28
73230-01	7 x 10 cm	4 x 7 cm	10	1/20
73230-02	8 x 15 cm	4.5 x 11.5 cm	10	1/20
73230-03	10 x 24 cm	6 x 20 cm	5	1/20
73230-05	10 x 35 cm	6 x 31 cm	5	1/10