

## ConMed Hyfrecator 2000


### Power Cord inlet instructions:

1. Note that the Hyfrecator 2000 has a black clamp at the power cord inlet socket. The clamp has three screws attached – picture below.




2. Clamp: The clamp has two dimples inside it on both sides. The purpose of the dimples is to squeeze on to the shoulders of the power cord and secure it within the socket (after the power cord is inserted fully into the socket and seated correctly. Turn the unit 'On' to verify power before tightening the three screws).

3. Connector from power cord. Note the shoulder.

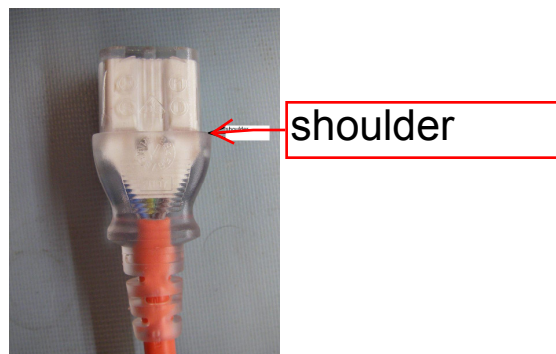

#### 4. Photo of a fully seated connector.

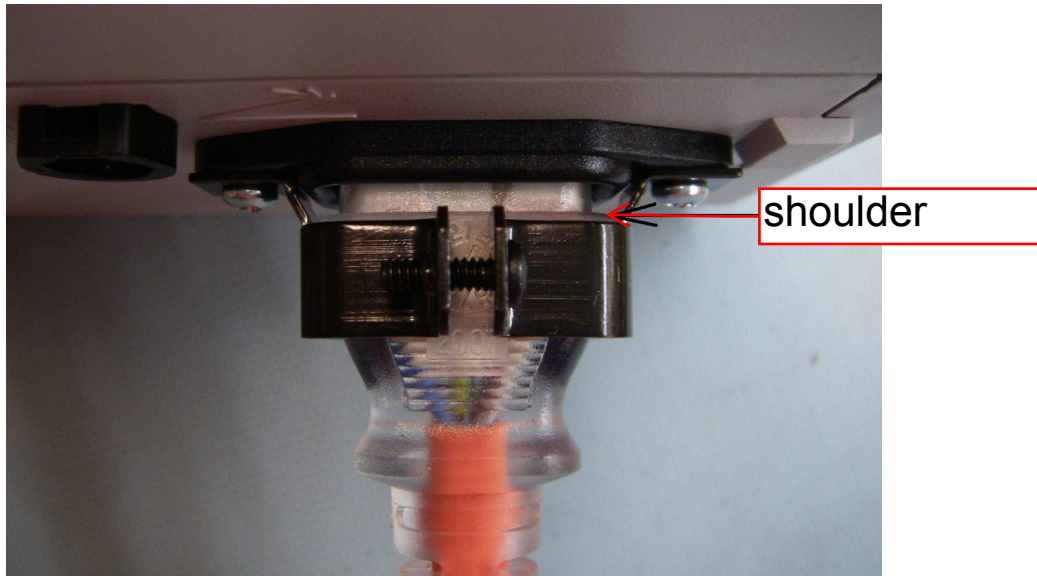

#### 5. Procedure:

- a. Loosen the top screw (black screw) all the way.
- b. Loosen both the other screws (important), just a little, about one turn is ok (otherwise the bracket doesn't open wide enough to allow the shoulder to pass through).
- c. Jiggle and push the mains cable connector in until the shoulder is visible inside the bracket and well seated within the socket.
- d. Turn the Hyfrecator 'ON' to verify power.
- e. Tighten the screws. Do the side ones first and then tighten the black screw to secure the clamp on the power cord connector.

Solid as a rock!

---

Supplied by

**team** medical  
supplies

✉ P.O. Box 6245 Dural DC, NSW 2158  
☎ 1300 22 44 50  
📠 1300 22 44 60  
@ info@teammed.com.au  
🌐 www.teammed.com.au