



## Hyfrecator® 2000 Power Settings Guide

Lesions	Power Setting (watts on low)	Type of Electrode
<b>BENIGN</b>		
Angiomas (cherry)	2-2.5	Sharp or blunt
Angiomas (spider)	2-2.5	Sharp or needle
Condyloma acuminata	12-18	Dull
Dermatosis papulosa nigra	2-2.5	Sharp or blunt
Pyogenic granulomas	16-20 or switch to high	Blunt
Sebaceous hyperplasia	2-2.5	Blunt
Seborrheic keratosis	10-14	Blunt
Skin tags (acrochordons)	2-2.5	Sharp
Syringomas	2-2.5	Sharp
Telangiectasias	2-2.5	Sharp or needle
Verrucae vulgaris	12-18	Blunt
Verrucae plana	12-18	Sharp or blunt
<b>MALIGNANT</b>		
Basal cell carcinoma	16-20	Blunt
Squamous cell carcinoma	16-20	Blunt

*Disclaimer: Every patient and every electrosurgical unit is different. These numbers are just suggestions and each clinician must find the best settings based on experience with their patients and unit.*

This table was published in *Dermatologic and Cosmetic Procedures in Office Practice*, vol. 1, Ed. Dr. Richard Usatine, Table 14-1, Chapter 14, Copyright Elsevier Inc, 2012.

Customer Service: 1-800-448-6506  
International Sales: 1-315-797-8375  
Fax: 1-800-438-3051

CONMED.com  
info@mail.conmed.com

525 French Road, Utica, NY 13502

©2016 CONMED Corporation, MCM2014007 Rev A 8/16