


# DISPOSABLE CURTAIN SELECTION GUIDE

**Haines®**  
MEDICAL AUSTRALIA

Use this guide to help you choose the right Haines® Disposable Curtains

## 1 CURTAIN LENGTH

Measure lengths of curtains required

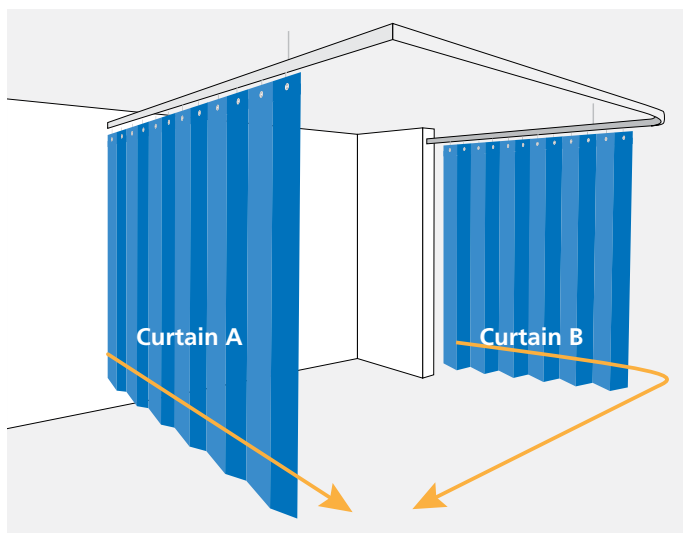


**Curtain A:** 4.5m length for short sections dividing cubicles

**Curtain B:** 7.5m length for long sections i.e. curved track at front of cubicle (2.5m length also available for small cubicles, privacy curtains, across windows - fits sizes 0.5m to 1.8m width areas)

## 2 CURTAIN DROP

Check the curtain drop required:

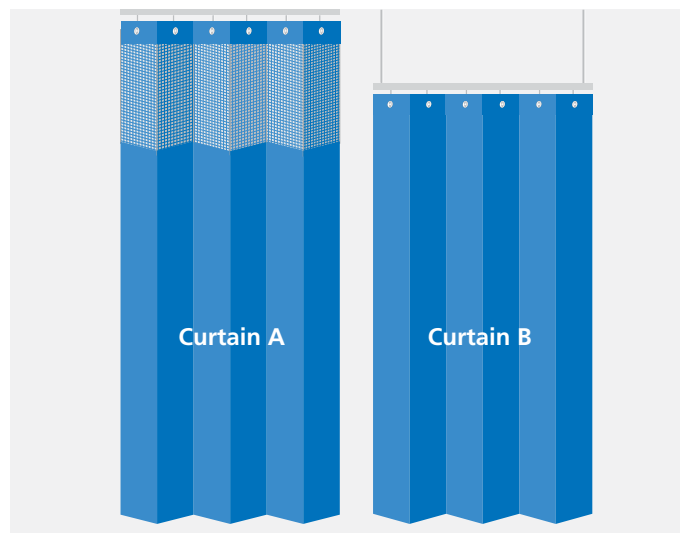


**Curtain A:** Track fixed to ceiling may require curtains with 50cm mesh top section to allow for sprinkler system. These curtains have a 2.3m drop.

**Curtain B:** Track suspended from ceiling does not require curtains with mesh top section. These curtains have a 2.0m drop.


## 3 HOOKS

If using existing hooks, check they slide easily around the track. If replacement hooks are required, Haines® Roller Hooks are an easy sliding option.

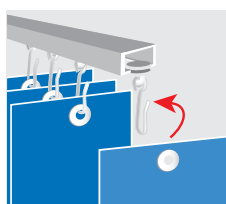


### Existing Hooks

When using existing curtain hooks, hook the Disposable Curtain eyelet onto your curtain hook

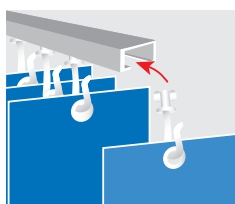

### Roller Hooks

For curtains that are difficult to slide along the track, use Haines® Roller Hooks for easy sliding.

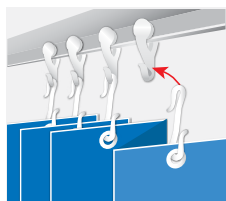

### "S" Type Hooks

Used to connect Disposable Curtains to existing curtain hooks. Ideal for window applications.

## 4 COLOUR

Choose your colour:

### Summer Blue



### Latte



### Cream

Ideal for hospital recovery areas



## 5 QUANTITY

Calculate the number of curtains and hooks required for your area/department/ward.

Width	Box Quantity	Packets of hooks required per box
2.5 metres	15	6
4.5 metres	8	5
7.5 metres	5	5



**HAINES® MEDICAL AUSTRALIA - SPECIALISTS IN INFECTION CONTROL AND PATIENT HANDLING EQUIPMENT**

26 Paranga Ave, Somerton Park  
South Australia 5044

T 08 8294 5999

F 08 8294 4337

E sales@hainesmedical.com.au

W hainesmedical.com.au

PRICES CURRENT FROM 30TH JUNE 2016



Disposable Curtains

