

❖ HEINE mini3000® Series – FAQ

1. How many mini3000 instruments are there?

There are 10 mini3000 instruments: 1. mini3000 otoscope, 2. mini3000 F.O. otoscope, 3. mini3000 LED F.O. otoscope, 4. mini3000 ophthalmoscope, 5. mini3000 LED ophthalmoscope, 6. mini3000 dermatoscope, 7. mini3000 LED dermatoscope, 8. mini3000 cliplamp/mini-c cliplamp, 9. mini3000 combilamp, 10. mini3000 tongue-blade holder. These instrument can be powered by either the mini3000 battery handle or the mini300 rechargeable handle plus miniNT charging station.

2. Which are the main advantages of the mini3000 series?

They are handy and ergonomic, with modern design; equipped with highest quality optics and illumination.

3. What are the highlights of LED^{HQ} in the LED mini-instruments?

Exceptional performance in the areas of light intensity, homogeneity, colour rendering, thermal management and durability.

4. What exactly distinguishes the HEINE XHL-technology in respective mini3000 instruments?

It's extremely homogenous with a CRI of over 98 and extremely good durability.

5. What is the „Performance-Indicator“ in the mini3000?

When instrument is turned on, it illuminates at maximum intensity. Then light is gradually reduced to the level that corresponds to current charging status of battery.

6. And what is meant by Fade-Out Function?

As opposed to conventional LED instruments, illumination output give warning before battery is empty and it doesn't just switch off. The illumination system slowly reduces brightness in accordance with charge remaining in battery. This ensures that examinations are not interrupted.

7. How often can the mini3000 instruments be switched on and off?

20.000 on-/off cycles are guaranteed. When returned into pocket it switches off automatically.

8. Which colors are the mini3000 instruments available in?

Black and blue.

