


3M™ Adaptic™ Digit Non-Adhering Dressing

Comfortable, easy to apply and remove

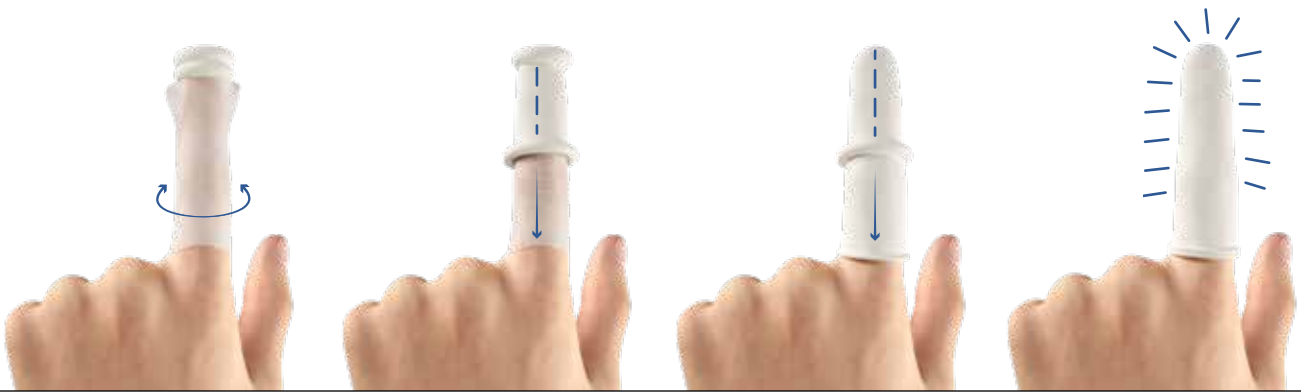

What is it?

3M™ Adaptic™ Digit Non-Adhering Dressing is made of a unique, tacky, silicone-coated fabric covered with a tubular bandage. It is specifically designed for dressing fingers and toes.



Comfortable

Adaptic Digit Dressing does not adhere to the wound. It is conformable and non-bulky, therefore allowing maximum digit flexibility.



Easy to apply and remove


Adaptic Digit Dressing is quick and simple to apply; no applicator or scissors are required. It removes cleanly from the wound, minimising trauma¹.

User Guide

Before application

Refer to the instructions for use. Select a size of 3M™ Adaptic™ Digit Non-Adhering Dressing according to the size of the finger or toe. Prepare the wound according to appropriate wound management protocol. Ensure skin surrounding the wound is dry. Remove the Adaptic Digit Dressing from the box.

Dressing preparation

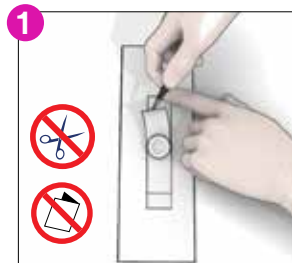

1. Using aseptic technique, peel apart the two sides of the packaging.
2. Remove Adaptic Digit Dressing from shiny backing strip.

Dressing application



1. Place primary silicone non-adhering layer over digit tip and gently smooth over any broken skin.
2. Roll down lower ring of tube bandage over primary strip (and secondary dressing if required).
3. Follow this with the top ring to form a double layered tube for protection.

Dressing change and removal



Dressing change frequency is dictated by good wound care practice and will depend on the condition of the wound. Adaptic Digit Dressing may be left in place for several days.

1. Gently roll up the upper ring of the tubular bandage and repeat with the other ring.
2. Gently remove the non-adhering layer from the wound bed and discard dressing.

Also available for toes



3M Company
2510 Conway Ave
St. Paul, MN 55144 USA

Phone 1-800-275-4524 (NPWT products)
1-800-228-3957
Web 3M.com/medical

Systagenix
part of 3M

Systagenix Wound Management Ltd
Gargrave, North Yorkshire
BD23 3RX, UK

When to use

3M™ Adaptic™ Digit Non-Adhering Dressing is indicated to manage a range of mild to severe digit injuries, such as:

- Lacerations and abrasions
- Traumatic injuries
- Nail extractions
- Suture lines



Adaptic Digit Dressing can be used both as a primary and secondary dressing².

Good to know...

In a clinical evaluation study

96%

of patients considered
**3M™ Adaptic™ Digit
Non-Adhering Dressing
easy to remove¹**



Ordering Information

3M™ Adaptic™ Digit Non-Adhering Dressing - Fingers

| Size | Dressings per box | Code | HCPCS Code |
|----------------------------|-------------------|---------|------------|
| Small 3/4 in (2cm) | 10 | MAD003U | A6457 |
| Medium 1 in (2.4cm) | 10 | MAD013U | A6457 |
| Large 1 1/8 in (2.8cm) | 10 | MAD023U | A6457 |
| Extra large 1 1/4 in (3cm) | 10 | MAD042U | A6457 |

3M™ Adaptic™ Digit Non-Adhering Dressing - Toes

| Size | Dressings per box | Code | HCPCS Code |
|------------------------|-------------------|---------|------------|
| Large 1 1/8 in (2.8cm) | 10 | MAD062U | A6457 |

To learn more about the benefits of **3M™ Adaptic™ Digit Non-Adhering Dressing** contact your 3M representative.

References

1. Whittaker S. A dressing for the occasion? A comparative trial of two dressings for digit injuries. Professional Nurse Magazine, 1994 Aug. 729-732. 2. Hargreaves, J, The Use of a Non-Adhering Digit Dressing on wounds in a Primary Care Setting. Paper presented EWMA, Krakow, Poland, May 2018.

Note: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. Rx only.

© 2021 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. 70-2013-1006-0 PRA-PM-US-00794 (05/21)