


ADAPTIC[®] DIGIT

For mild to severe digit injuries

- Comfortable¹
- Convenient to apply¹
- Easy to remove¹

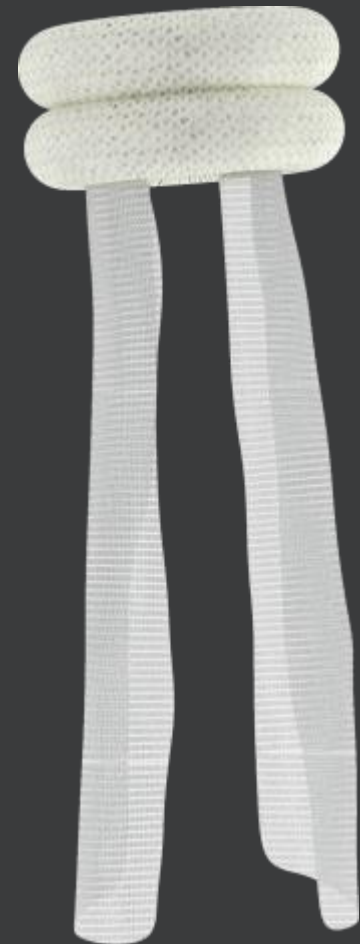

1. Whittaker, S. A dressing for the occasion? A comparative trial of two dressings for digit injuries. Professional Nurse Magazine, 1994, 729-732.

Key features and benefits

ADAPTIC® DIGIT Non-Adhering Digit Dressing

ADAPTIC® DIGIT is made of a unique tacky silicone coated open-knit cellulose fabric covered by a double-coiled tubular bandage that is specifically designed for dressing fingers and toes:

ADAPTIC® DIGIT

- Is a quick and convenient dressing for mild to severe digit injuries¹
- Is conformable and non-bulky, therefore allowing maximum digit flexibility¹
- Removes cleanly from the wound, minimising trauma and pain at dressing removal¹



Demonstrating ADAPTIC® DIGIT Non-Adhering Dressing

ADAPTIC® DIGIT Non-Adhering Digit Dressing is best demonstrated by showing how easy it is to apply.

Dressing Application:

1. Remove shiny backing strip.
2. Place primary non-adhering layer over digit tip and gently smooth to cover any broken skin.
3. Roll down lower ring of tube bandage over primary strip (and secondary dressing if required). Follow this with the top ring to form a double layered tube for protection.
4. No applicator, no scissors or tape are needed to apply ADAPTIC® DIGIT.

