
Infusion Set Volumat Line

REF M46444600S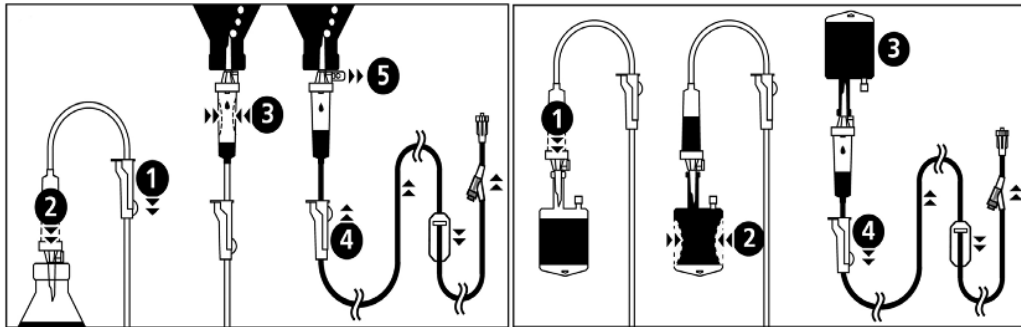

de - Gebrauchsanweisung / **en** - Instruction for use / **fr** - Mode d'emploi / **bg** - Инструкции за употреба / **cs** - Návod k Použití / **da** - Brugsanvisning / **el** - Οδηγίες Χρήσης / **es** - Instrucciones de uso / **fi** - Käyttöohje / **hr** - Upute za uporabu / **hu** - Használati Utasítás / **is** - Notkunarleiðbeiningar / **it** - Istruzioni per l'uso / **nl** - Gebruiksaanwijzing / **no** - Bruksanvisning / **pl** - Instrukcja obsługi / **pt** - Instruções de utilização / **ro** - Instrucțiunile de Utiliza re / **ru** - Инструкция по применению / **sk** - Návod na použitie / **sl** - Upute za uporabu / **sv** - Bruksanvisning / **tr** - Kullanma kılavuzları

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en - Special recommendations linked to the use of Volumat Lines: The filter size and the materials used in the infusion set are mentioned in its individual packaging: check they are suitable with the fluid or drug to be administered. The infusion set may be equipped with ports (K-NECT needle-free access, or 3-way stopcock). These ports should be accessed by respecting aseptic procedures and the pump is on hold. The upstream ports (above the pump) should not be used to deliver a manual bolus into the line. They should be used only to connect a secondary line. The downstream ports (below the pump) may be used to administer a manual bolus by means of a luer lock syringe into the line: when administering a bolus, we recommend to hold the infusion. Use of the Volumat Lines with two spikes or with secondary lines: we do not recommend to keep the two lines open. Only the line from which the fluid is supposed to be administered should be open, the others must not be open. When a Gravity Infusion Set shall be connected to the downstream port make sure that the pump is switched off or the Set protected by a Back Check Valve.

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Fresenius Kabi AG