REF M46444600S

CE 0123

STERILE EO

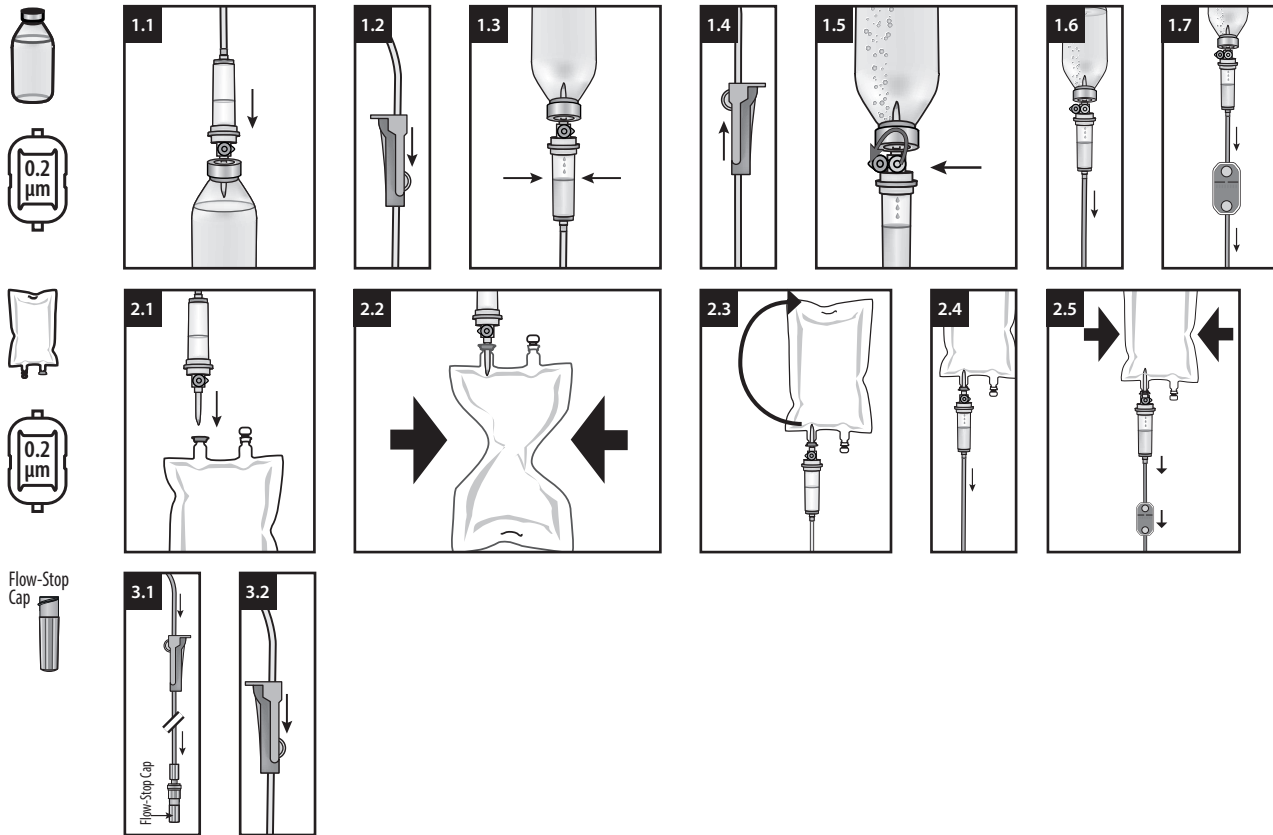
ifu.fresenius-kabi.com

~265 cm

~27,8 ml

6,8 ml/m (40 °C)
7,6 ml/m (40 °C/1,5 bar)

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en - Instruction for use

Intended Purpose

IV administration set for infusion pumps.
Affected persons are patients undergoing IV treatment. Use depends on the patient's condition and is in the responsibility of the qualified health care professional.

Handling Guidelines

The infusion sets can be used for no longer than 96 h for all solutions except lipid solution. For the infusion of lipid solution the infusion sets can be used for no longer than 24 h. The Volumat administration sets shall be used exclusively on Volumat Agilia, Agilia VP and Exelia VP pumps or by gravity.

inline filter

The filter size used in the infusion set are mentioned in its individual packaging. Check if filter is suitable with the drug or fluid to be administered.

The PTFE air vent material may lose its hydrophobic properties or "wet out" when it is wet by a liquid with a surface tension close to or less than 27 mN/m, e.g. disinfectants such as 70% Isopropyl Alcohol.

Safety and Warning Information

- Do not use if protective caps are displaced.
- Some drugs might be incompatible with tubing materials and components or parallel infused drugs. Please refer to SMPC of drugs.
- Please refer to the instructions for use for the Volumat Agilia, Agilia VP and Exelia VP pumps.
- Reuse creates a potential risk of patient or user infection. Contamination of the device may lead to injury, illness or death of the patient. Reprocessing may compromise the structural integrity of the device.

Disposal

Bio-hazardous waste must be disposed in accordance with the local protocol.

Reporting of Incidents

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority.

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Fresenius Kabi AG