Cleaning Indicators

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REF EC.WASHTEST

Results reference guide (✓OK / ✗FAIL)							
Visual results	Location inside washer	Water pressure	Mechanical functions (detergent injection pump, spray arms, etc)	Temp.	Exposure time	Detergent (type, dose)	Water quality
	✓	✓	✓	✓	✓	✓	✓
	✗	✗	✗	✗	✗	✗	✗
	✗✗	✗✗	✗✗	✗✗	✗✗	✗✗	✗✗
	✗✗✗	✗✗✗	✗✗✗	✗✗✗	✗✗✗	✗✗✗	✗✗✗
	—	—	—	—	—	—	—

NOTE: These results could be a consequence of failure in one or a combination of variables. When detecting the failed variable/s, make the corrective actions. If the problem persists, contact the technical service of your washing machine. The higher the quantity of **X**, the higher the influence of that variable on the failure of the cycle. Reference colors exhibited in printed prospects as well as those displayed in our website and soft copies of other documents, do not necessarily represent the real color as shown in the actual indicators.

Cleaning Indicators

For cleaning performance tests

English

Control of cleaning and washing processes of medical instruments and materials is highly important since the outcome affects subsequent disinfection and/or sterilization processes success. Cleaning control is particularly relevant in automatic washing machines, where even though process parameters may be well controlled, the cleaning final result depends largely on uncontrolled factors. Washing efficacy is influenced by detergent, water temperature, time and washing penetration. A good cleaning result is achieved by an appropriate combination of these variables.

Product description

ECS® EC.WASHTEST indicators consist of a 47 x 100 mm temperature stable support containing a mixture of specially combined natural colored components (proteins, carbohydrates, fatty acids, and dyes). In all cases, indicator colored test soil is non-toxic, dissolves with most of the cleaning agent and is swept along with the washing fluid. EC.WASHTEST cleaning indicators formula allows detection of all the factors that affect the outcome of the washing process. EC.WASHTEST indicators can be used for routine control of cleaning processes and allow monitoring of washing effectiveness in automatic machines with different washing configurations. The indicators efficiency is validated by comparing against the test soils detailed in ISO/TS 15883-5:2005 standard annexes.

Indicators Classification

Code	Sample color	Annex of standard	Challenge level	Recommended use
EC.WASHTEST	Fuchsia	Q	Very high	Temperature > 50 °C Cycle duration > 5 min.

NOTE: We recommend placing several identical indicators at different positions inside the chamber of the washing machine or at least one indicator in every cycle in different positions, so as to monitor the performance of all areas of the machine during the washing process.

Characteristics

EC.WASHTEST indicators are placed within a special holder. Holder design mimics a realistic challenge of thoroughly cleaning hard-to-reach areas to the washing disinfecting process. When EC.WASHTEST is placed within the holder, the test soil is covered by a mesh, thus posing a difficulty for water stream to reach it, mimicking shadowed or occluded surfaces. EC.WASHTEST indicators provide constant cleaning characteristics compared to the use of natural samples based assays which might suffer modifications of adhesion and polarity properties during storage time. The EC.WASHTEST system provides high confidence and high reliability to reveal defects in the cleaning process.

Instructions for use

- 1 - Place the indicator on the holder.
- 2 - Place the system on the tray along with the material.
- 3 - Once the cleaning cycle is over, remove the indicator.
- 4 - Analyze EC.WASHTEST indicators using the reference table and record results. If unfavorable results are obtained, perform the appropriate corrective actions and repeat the washing process.

Use at least one EC.WASHTEST indicator for each cleaning cycle.

Precautions

- Do not reuse EC.WASHTEST indicators.
- Do not touch the indicator ink with your hands.

Storage

EC.WASHTEST indicators should be stored at temperatures between 10-30 °C, protected from light and moisture, within its original packaging. **NOTE:** in case of remaining traces of indicator colored spot after the washing process, do not store the used indicator under direct light since this may deteriorate its coloring.

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