



Gynaecology Table

Assembly Instructions

There are three different configurations for the Gynaecology Table. Below are separate instructions on how to assemble the table to suit your requirements

Tip - prior to completing any of these tasks it is recommended that you raise the height of the bed and tilt the seat.



Foot Pad Configuration

Please note these steps will need to be repeated from each side of the table.



Step 1

Remove the handles from the bed - to do this loosen the screw knob at the bottom of the handle.

Unhook the handle from the rail.

You do not need the handles when using the foot pads.



Step 2

On the rail where the handles were located there is a screw at each end. Undo and remove one screw and loosen the other.

Loosening the screw you didn't remove will make Step 3 easier.



Step 3

The small block attached to the rail is facing the wrong direction. It is packaged this way to prevent damage in transit. Loosen the screw knob under the block.

Slide the block along the rail to remove from the rail.



Step 4

Return the block you removed in Step 3 by sliding it onto the rail. Make sure the hole is facing outwards and the screw knob is facing the bottom of the table.

Tighten the screw knob to lock the block in place.

Screw the rail back on.



Step 5

There is a large hole on the block insert the footpads into this hole.

There should be additional screw knobs in the box you received, insert the screw knob into the smaller hole on the side of the block

Tighten to lock the height of the foot pad in place.



Please note: you only need to complete steps 2, 3 and 4 the first time you use the foot pads.



Gynaecology Table

Assembly Instructions

Attaching the Stirrups

Please note these steps will need to be repeated from each side of the table.



Step 1

At the front of the seat section of the table you will find two rectangular powder coated rods.

Loosen the screw knobs under the rod. Pull out the rods until they extend past the edge of the seat or desired length.

Re-tighten the screw knob to lock the rod in place.



Step 2

At the front of the rod there is a hole with screw knob.

Loosen the knob and then insert the stirrup into the hole.

Re-tighten the screw knob to secure the stirrup.



Attaching the Leg Extension



Step 1

At the front of the seat section of the table you will find two rectangular powder coated rods, these are for the stirrups (see above).

Loosen the screw knobs under the rod and fully remove the rods from the bed.



Step 2

There should be a separate padded leg extension in the box.

At the end of the leg extension there will be two powder coated rods.

Insert the leg extension rods into the holes where you removed the stirrup rods from.

Re-tighten the screw knobs to secure the leg extension.



Watch the product specification video online at www.pacificmedical.com.au

Pacific Medical

Phone: 1300 076 332

Email: info@pacificmedical.com.au

www.pacificmedical.com.au



Supplied By:



Team Medical

Phone: 1300 22 44 50

Fax: 1300 22 44 60

www.teammed.com.au

info@teammed.com.au