

# **CLUNGENE® COVID-19 Antigen Rapid Test**

For Australia Only






Hangzhou Clongene Biotech Co., Ltd.



# COVID-19 Antigen Rapid Test (For Self-testing)

Sample: Nasal swab

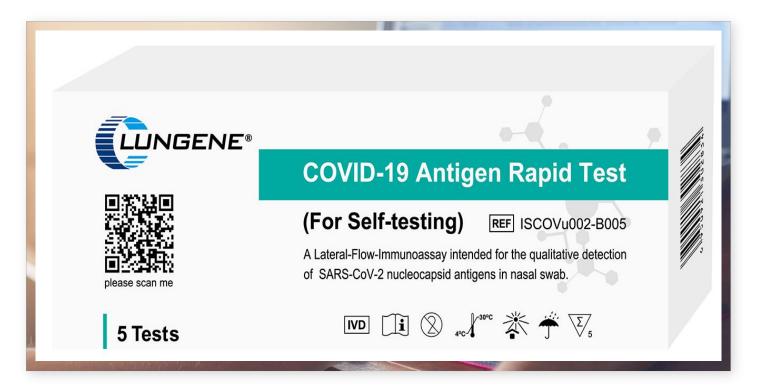


#### **Product Features**

- \* For Self-testing
- \* Very high sensitivity
- \* Get results at 15-20 minutes
- \* No need instrument



For more information please scan the QR code



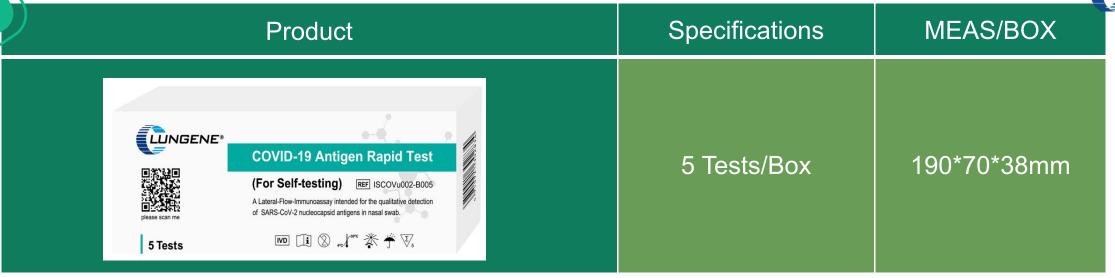

Test cassettes	Extraction reagent tubes *5	Swabs	Waste bags	Instructions for Use
*5		*5	*5	*1
** On the Conclusion (see Conc.)  **On the Conclusion (see Conc.)				Concerned:  Conference of the Antiggan Regal Test  Concerned of the Concer


Packing Carton I	Information
------------------	-------------

MEAS/CTN	TESTS/CTN	Estimated G.W./CTN
59*40*58cm	1200tests	21.0kg




## Preparing to do the test

Open the box and you will get the components shown below.

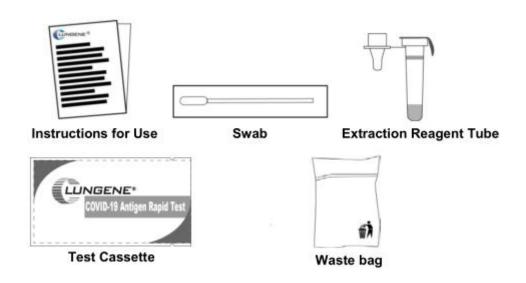

Wash your hands in soapy water and dry thoroughly.

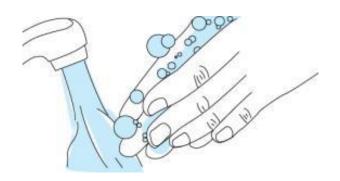





## **Step-By-Step Instructions**




1. Open Extraction Reagent Tube

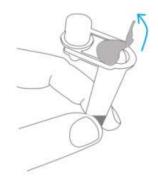


2. Insert Tube into Box

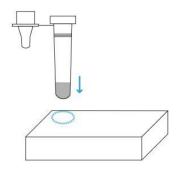


3. Remove the Swab

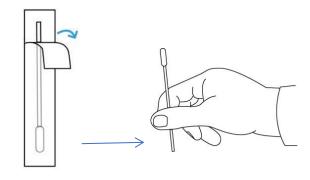


4. Swab the Left Nostril

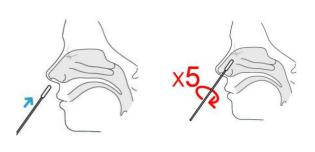


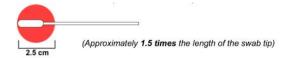

5. Swab the Right Nostril

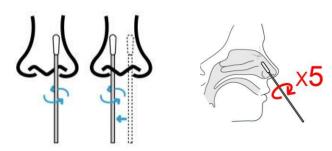


6. Insert the Swab into the Tube

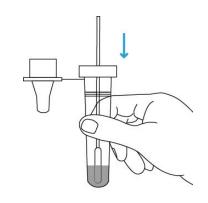




## **Step-By-Step Instructions**


#### 7. Rotate the Swab 5 Times

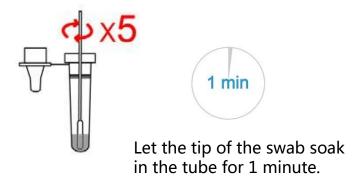

#### 8. Remove the Swab

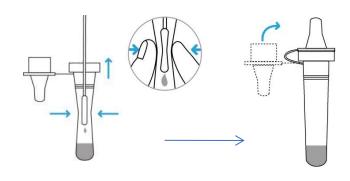


#### 9. Take out the Test Cassette from the pouch

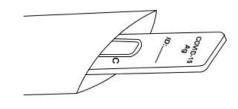

#### 10. Add Sample to the Sample Well



#### 11. Start the timer and wait 15 minutes.

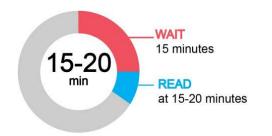


Read the test result at **15-20** minutes.

**DO NOT** read the result after 20 minutes.



### Read result

## Dispose of the used test kit

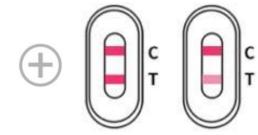

#### **Positive Result**

Two lines appear.



#### **Negative Result**

One coloured line appears at the control region (C), and no line appears at the test region (T).



#### **Invalid Result**

Control (C) line fails to appear.



Collect all parts of the test kit and place in the waste bag that can be placed in the general waste.



### Clungene Covid-19 Antigen Rapid Test (for self-testing)

#### Listed with TGA under ARTG No. 333341

#### COVID-19 self-tests (home use) approved by TGA

\*Clicking on the name of the test will link to the instructions on how to use it.

PDF Instructions on how to use the test are provided as a supplementary resource for the visually read tests listed below and do not meet disability access requirements.

Search:

Show	100	~	entries

Name of self-test* and how to use the test Specimen type	Australian Sponsor \$ (supplier)	Manufacturer <del>\$</del>	ARTG \$	Clinical Sensitivity <b>‡</b>	Date Approved
Clungene Covid-19 Antigen Rapid Test for self-testing (pdf,703kb) Nasal swab	APAC Security Pty Ltd	Hangzhou Clongene Biotech Co Ltd (China)	333341	Very high sensitivity	7 January 2022

Manufacturer: Hangzhou Clongene Biotech Co Ltd	Self-test
Name of test: Clungene Covid-19 Antigen Rapid Test for self-testing	Antigen Test
Australian sponsor: APAC Security Pty Ltd	
Date approved for supply: 7 Jan 2022	
Manufacturer: Hangzhou Clongene Biotech Co Ltd	Point-of-care test*
Name of test: Clungene Covid-19 Antigen Rapid Test	Antigen Test
Australian sponsor: APAC Security Pty Ltd	-1000 <del>-2</del> 00-100
Date approved for supply: 7 Jan 2022	
Manufacturer: Hangzhou Clongene Biotech Co Ltd (China)	Point-of-care test*
Name of test: COVID-19 IgG/IgM Rapid Test Cassette	Lateral Flow IgG/IgM
Australian sponsor: APAC Security Pty Ltd	
Date approved for supply: 26 Mar 2020	