

SPEEDY CLEAN

HOSPITAL GRADE ANIONIC NEUTRAL DETERGENT AUSTRALIAN MEDICAL DEVICE CLASS 1 ARTG NO. 125529

SPEEDY CLEAN is an excellent ready-to-use, hospital grade, anionic neutral detergent which leaves a streak free finish. SPEEDY CLEAN uses Whiteley's Emulsion Systems Technology, which enhances SPEEDY CLEAN's ability to remove oils and dirt that are encountered in everyday cleaning.

SPEEDY CLEAN is ideal for cleaning surfaces and equipment in the healthcare environment. The single-use container improves infection control by avoiding possible contamination. SPEEDY CLEAN is quality assured as Whiteley is an accredited company under ISO 13485:2016. SPEEDY CLEAN is non toxic and safe for use on most surfaces.

SPEEDY CLEAN is a Class 1 Medical Device and can be used to clean other Medical Device surfaces (such as Theatre trolleys etc).

SPEEDY CLEAN is:

- pleasant, powder fragranced
- ready to use and is conveniently packaged in a 500mL squeeze bottle
- designed to not cause staining on fabrics or carpets
- pH neutral and non-toxic
- friendly to metals (non corrosive) and safe to use on most surfaces

SPEEDY CLEAN has been formulated to replace most cleaning products. Do not mix with chemicals.

INFECTION CONTROL:

Healthcare providers must be confident that detergents and wipes used for cleaning are free of bacterial contamination and are labelled correctly with all relevant safety information.

SPEEDY CLEAN aids in the Infection Control process because of its single use package. Coupled with Viraclean® this makes for simplification of the cleaning and disinfection process.

SPEEDY CLEAN can be used on all surfaces in the following areas:

- Nursery
- CSSD
- Intensive Care Units
- Theatre Equipment
- Theatre tables and benches
- Ward areas
- Nurses' station

ABOUT LMS:

2**Hospital Grade Neutral Detergent**

Our comprehensive learning platform offers a broad range of information including the latest information on Infection Prevention and Control, recommended products and cleaning processes for individual markets.



Scan to register for Whiteley's Online Learning Centre



CHEMICAL COMPOSITION AND PROPERTIES:

Surfactant system:	Blended
Colour:	Pale Blue
pH:	Neutral
Solubility:	100% in water
Specific Gravity:	1.00
Shelf Life:	15 months
Safety Equipment:	Gloves recommended

SAFETY DATA: Information is listed in the Safety Data Sheet.

PACK SIZE: 12 x 500mL

PROFESSIONAL CLEANING COLOUR CODED GUIDE:

The Industrial product range has been streamlined to make it easier for you to choose the right product. This provides a simple, straightforward guide for finding products for specific cleaning needs. The product range has been categorised into an easy-to-use product selection chart.

To view the Professional Cleaning Colour Coded Chart visit www.whiteley.com.au/professional-cleaning-colour-codedguide

HEALTH AND SAFETY STATEMENT:

There is no way of predicting an individual's response to cleaning or disinfecting products. All cleaning and disinfecting products have potential to cause skin and other irritation in susceptible individuals. Direct physical contact with the skin should be avoided. If irritation does occur when using this product or any other cleaning/disinfecting solution then the individual should be removed from direct contact with the solution until a satisfactory safe working environment for that individual is determined.

ENVIRONMENTAL STATEMENT:

Whiteley manufactures products that comply with Australian and International environmental policy. Whiteley aims to ensure that every product is 100% biodegradable, whilst achieving optimal performance and benefit to customers. All Whiteley products are supplied in 100% recyclable containers. Whiteley and its staff are fully committed to ensuring that our environment and those international environs in which our products are used are enhanced by the use of technology, and that these environments are preserved for the enjoyment of future generations.

© 2022 This document remains the property of Whiteley Corporation Pty Ltd.

Alterations are not permitted without prior written authorisation from Whiteley Corporation Pty Ltd.

2 Hospital Grade Neutral Detergent

