

# + WYNSTAN™ HEALTH



## Level 2 Surgical Disposable Face Mask with Ear Loops Product Code WH404101 ARTG no. 364810

### Product Overview

AS 4381:2015 Level 2 rated surgical disposable face mask designed for use in settings where minimal blood/droplet exposure is likely. This

includes changing dressings on small or healing wounds in primary care or the emergency department and for use in dentistry procedures.

### Product Features

- AS 4381:2015 Level 2 rating for:
- Differential pressure ( $< 5.0 \text{ mm H}_2\text{O}/\text{cm}^2$ )
- Resistance to penetration by synthetic blood (120 mmHg)
- Bacterial filtration efficiency ( $\geq 98\%$ )
- High quality design and construction
- Comfortable four-layer filtration with good breathability
- Wide and firm adjustable nose bridge to reduce fogging
- One size fits all
- Latex free
- Proudly Australian designed, manufactured and owned

### Fitting Instructions

1. Place mask across the bridge of your nose and attach ear loops. Ensure the mask covers your nose, mouth and chin
2. Press the nose clip comfortably and evenly across the bridge of the nose to ensure a good seal
3. Press the mask firmly onto the face
4. Gently pull down the base of the mask until you create a close fit to both sides of your face
5. Follow appropriate Health Care Institution surgical face mask "fit & check" protocols
6. This face mask can be used continuously for up to 4 hours, as long as the mask does not become moist, soiled or damaged
7. Where single-use face masks are used for extended periods and are needed to be removed to eat and/or drink, hand hygiene and sanitizing should be maintained
8. This face mask must not be stored or reused after removal

# + WYNSTAN™ HEALTH



## Level 2 Surgical Disposable Face Mask with Ear Loops Product Code WH404101 ARTG no. 364810

### Product Construction

- Outer layer: 100% Spunbond Polypropylene
- Middle layer 2: 100% Spunbond Polypropylene
- Middle layer 1: 100% Melt Blown Polypropylene
- Inner layer: 100% Spunbond Polypropylene

### Product Storage

Product should be stored in a clean dry environment away from direct sun light under 30 degrees Celsius

### Packaging Details

- Individual Box: 50 Masks
- Bulk Box: 600 Masks (12 Individual Boxes)
- Pallet: 28,800 Masks (48 Bulk Boxes)



#### Team Medical Supplies

Phone 1300 22 44 50

Fax 1300 22 44 60

info@teammed.com.au

www.teammed.com.au



#### BP Medical (WA)

Phone 1300 33 55 60

Fax 1300 33 55 70

info@bpmedical.com.au

www.bpmedical.com.au