

# AED PLUS® BATTERY AND ELECTRODE REPLACEMENT



## Replacing CPR-D® Padz electrodes:

- 1 Ensure that new electrodes have same reference or part number as used or expired electrodes.
- 2 Connect electrode connector to AED Plus.
- 3 Electrode connector must be firmly seated into AED.
- 4 Ensure that electrodes always remain connected to AED so device is ready in case of emergency.
- 5 Place electrodes with rescue kit onto face of AED and install AED lid over top, with expiration date tag exposed through slot on lid.



## Replacing batteries:

- 1 Remove back cover of AED Plus to expose batteries.
- 2 Remove old batteries from AED and discard in accordance with local regulations.
- 3 Wait 2 minutes to allow any static electricity to drain from AED.
- 4 Place new batteries lightly into their slots (do not push them in fully), ensuring that positive and negative polarities are correctly positioned. Batteries should not make full contact at this time.
- 5 Push left 4 batteries into their slot first, followed by right 4 batteries, and finally 2 batteries in middle.
- 6 Listen for device voice to prompt you for next action.
- 7 AED will detect electrodes first.
- 8 AED will prompt you to press blue button inside battery compartment. Do so within 10 seconds only if batteries are new. This will reset battery timer for new batteries.
- 9 Replace battery cover.
- 10 Indicator on front of AED should now show a green check mark.