

MC600 SUCTION UNIT CLEANING INSTRUCTIONS

Lines & Cannister

Step 1:

Once the patient load is finished for the day, make up 500ml of diluted clinical detergent in a container (we recommend Clinidet or Sonidet).



Step 2:

Turn on the suction machine and place the patient tube with suction controller attached into the container of diluted detergent.

This will pull the detergent through the suction lines and deposit it and any debris from the tubes into the suction canister.



Step 3:

Remove the lid and take the suction canister to dirty sink .

Wash out debris and cleaner from canister.

Rinse canister with clean water and leave 200ml of clean tap water in the bottom of the canister for the next day.



Machine External Surfaces

Machine surfaces must be cleaned with a detergent impregnated wipe eg. housing, switches.



Supplied by

team medical
supplies

✉ P.O. Box 6245 Dural DC, NSW 2158

☎ 1300 22 44 50

☎ 1300 22 44 60

@ info@teammed.com.au

🌐 www.teammed.com.au



Ph: 1300 099 663 Fax: 1300 099 300
Email: info@zonemedical.com.au
7/22 Mavis Court, Yatala, Qld, 4207
www.zonemedical.com.au