

ELECTRONICA PC AUDIOMETER




Quick Start Guide

What is included



Where the plugs go

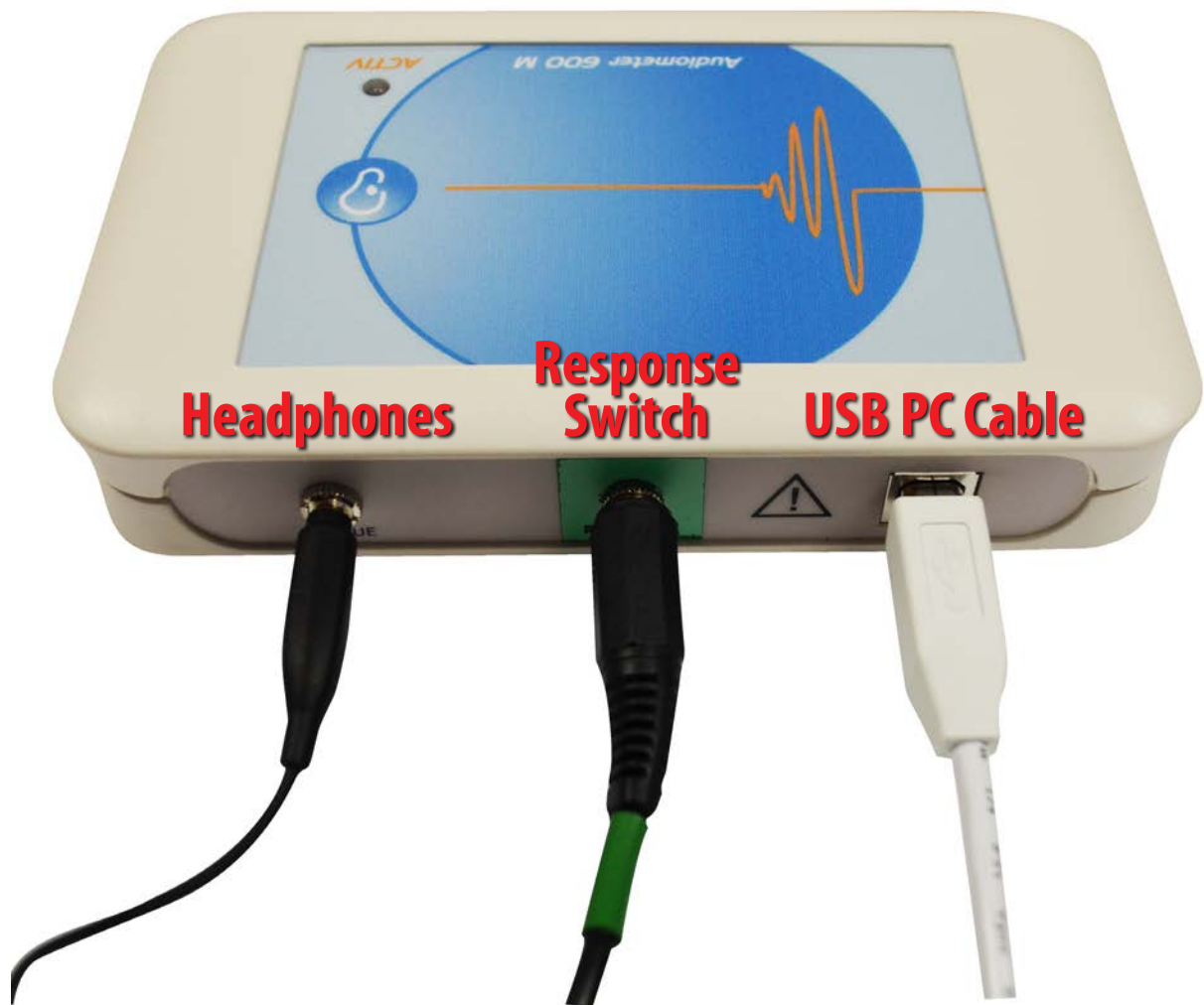

Electronica PC Audiometer | Quick Start


1. Insert CD and Click 'Install 600M software'




2. Click next to begin install




3. Select the destination to install software
(recommend default program files)

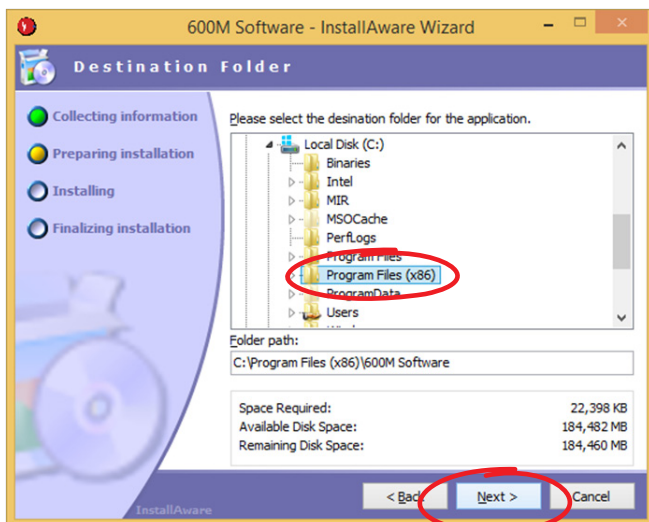


4. Wait for the software to install

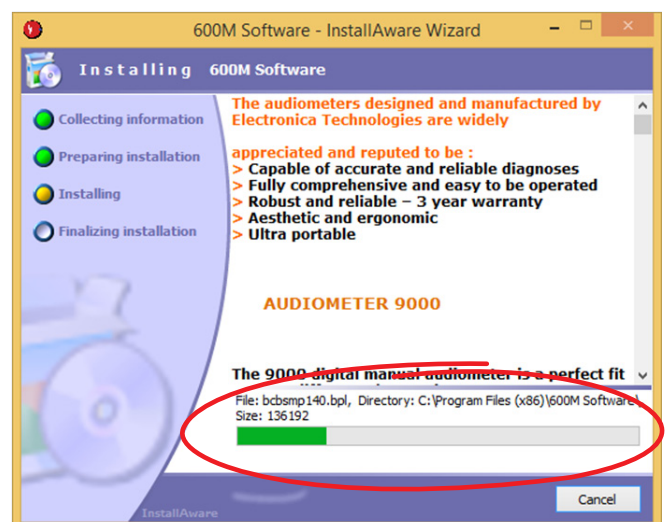


5. You have now successfully installed the 600M software. Click finish to run the program

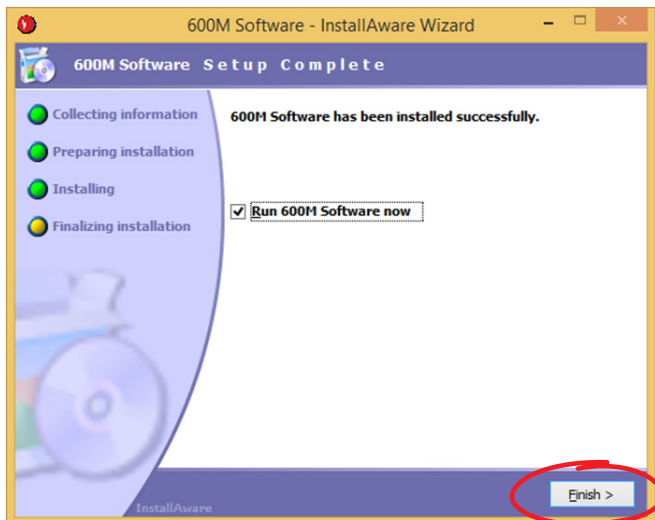


6. Leave default Operator and click OK. This can be changes at a later date

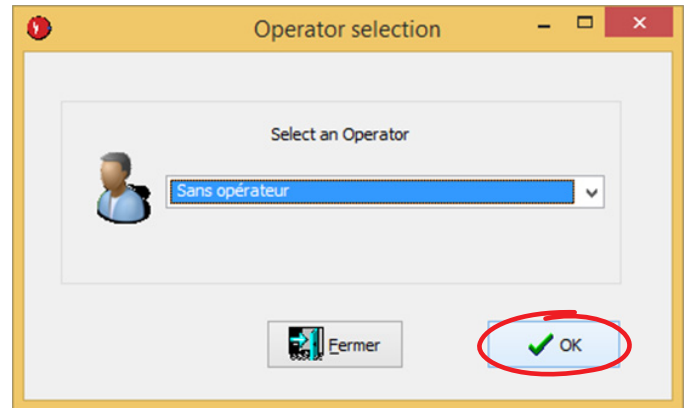

7. Create a new patient

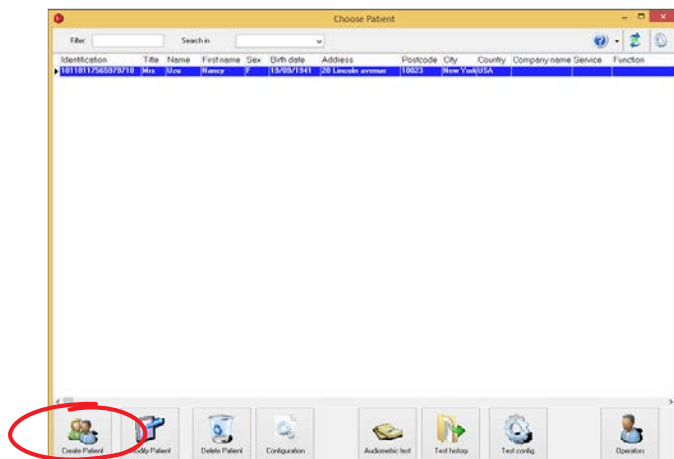


8. Add Necessary Patient details and click OK

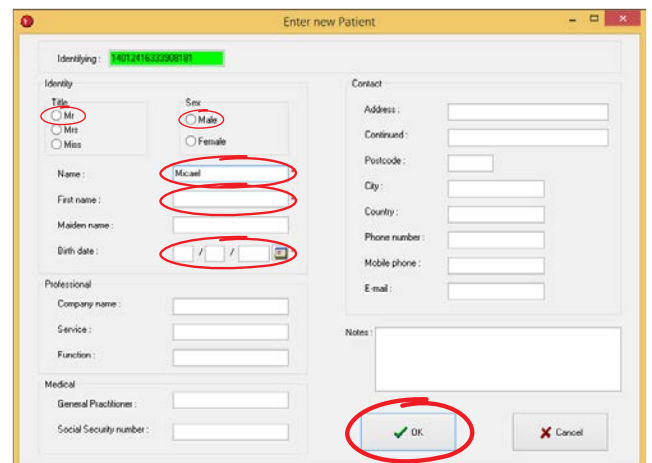


Electronica PC Audiometer | Quick Start


9. Select your new Patient and click Audiometric Test

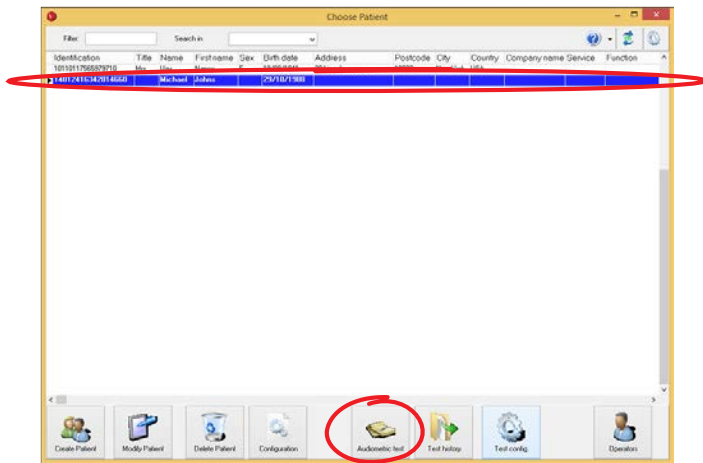


10. Adjust preferences for test and press Test Launch to begin test

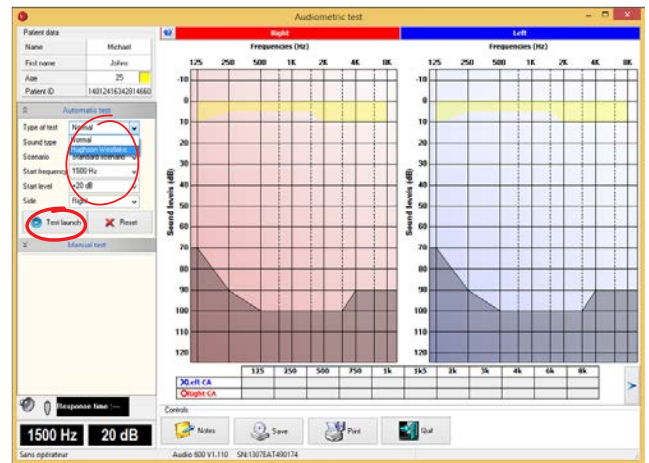


You can pause the test at any time by clicking the pause/stop button.

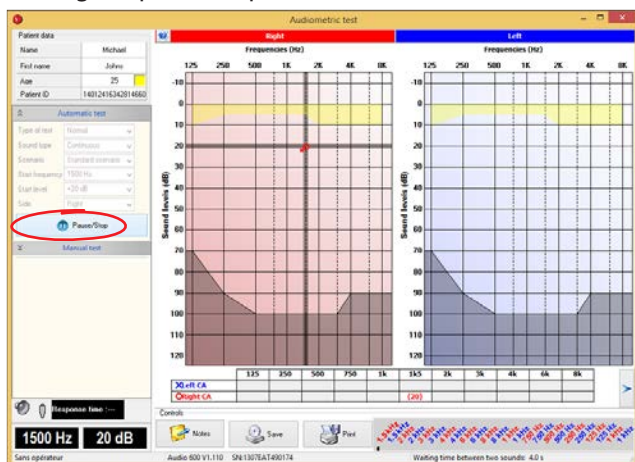

Continue test until finished.

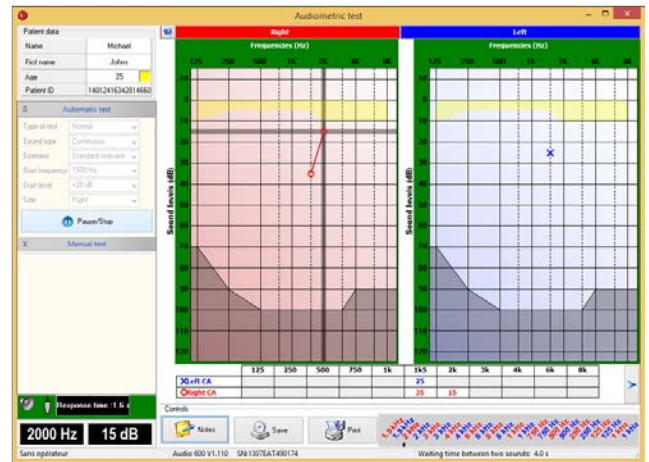

The test is now complete. You can save or print the report.

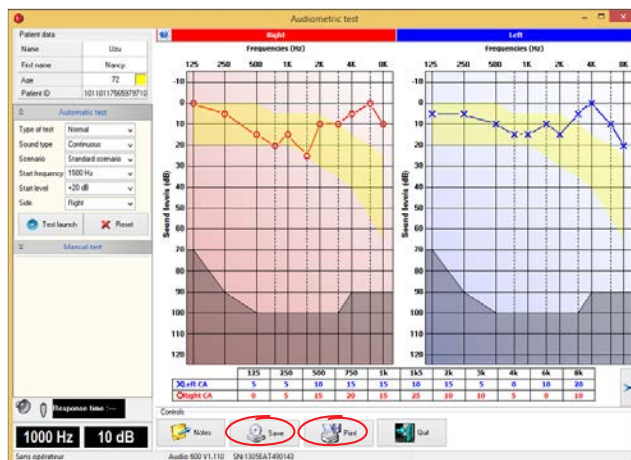

The completed PDF report can be uploaded to patient history.

Sans opérateur

Patient data **Uzu Nancy**

Birth date: 19/09/1941
 Age: 72
 Company name:
 ID: 10110117565979710

Date of the test: **28/01/2014 4:49:19 PM**
 Duration: 10mn 15s
 Type of test: Auto CA
 Normal
 Continuous
 Standard scenario

Audiometer

Type: Audio 600 V1.110
 Serial number: 1305EAT490143
 Calibrated on: 14/05/2013

Right ear

Left ear

Frequencies (Hz)	125	250	500	750	1k	1k5	2k	3k	4k	6k	8k
Left CA	5	5	10	15	15	10	15	5	0	10	20
Right CA	0	5	15	20	15	25	10	10	5	0	10

Calculation

	PAM	IPA
Left CA	13.8 dB	5.0 dB
Right CA	13.8 dB	5.0 dB

Notes

Logiciel 600M

29/01/2014 9:29:44

